



# Quick Start Guide

OSSIX® Plus


OSSIX® Volumax

OSSIX™ Bone


Find out more on  
[dentsplysirona.com](https://dentsplysirona.com)

 **Dentsply  
Sirona**  
Implants

## OSSIX® Plus

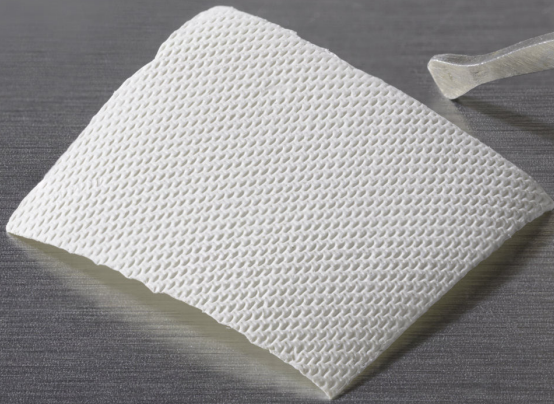

## OSSIX® Volumax

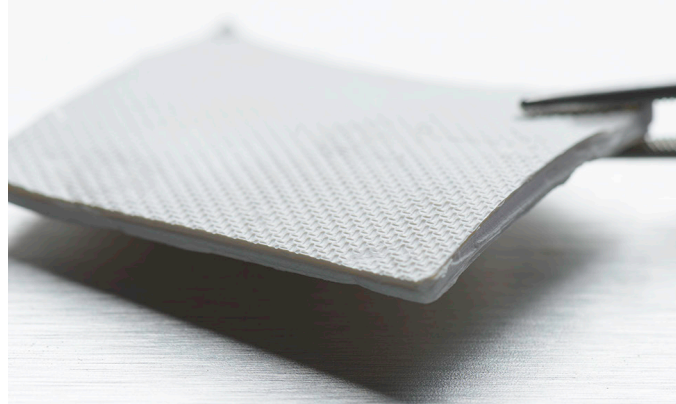

## OSSIX™ Bone



Hydrate in saline 30 seconds



Can be used on either side

Trim after wetting to avoid damaging membrane



Lay in place; do not suture through the membrane; can layer if necessary



1 mm space between adjacent teeth & membrane



Use periosteal suture to hold in place if needed



Hydrate in saline 30 seconds



Can be used on either side

Trim after wetting to avoid damaging scaffold



Can be folded if more volume is desired



Lay in place; do not suture through the scaffold



Use periosteal suture to hold in place if needed



If needed, trim carefully prior to hydration



Hydrate with patient's blood until completely saturated



Will expand when hydrated to fill socket



Gently press into defect until level with crest of bone; do not suture through the block



Align with crest of natural bone to achieve proper results; do not overfill or condense



Suture over OSSIX Bone in the socket to hold in place, if necessary

## Preferred Indications

- GBR and GTR procedures
- Ridge augmentation
- Cases with challenging primary closure; high risk of exposure or deliberate exposure

## Preferred Indications

- Dehiscence type defects
- Ridge preservation
- Cases where hard tissue volume is needed

## Preferred Indications

- Extraction socket preservation
- Sinus elevation (crestal approach)
- Lateral/vertical augmentation (with OSSIX Plus membrane); Periosteal fixation advised